Cosmetic tooth whitening for use at home	EN	2
Sbiancamento dentale cosmetico per uso domiciliare	IT	5
Kosmetische Zahnaufhellung für den Hausgebrauch	DE	8
Blanchiment dentaire cosmétique à domicile	FR	11
Blanqueamiento dental cosmético para uso domiciliario	ES	14
Branqueamento dentário cosmético para uso doméstico	PT	17
Cosmetische bleekbehandeling voor thuisgebruik	NL	20
Kosmetisk tandblegning til hjemmebrug	DA	23
Kosmeettinen hampaiden valkaisu kotikäyttöön	FI	26
Kosmetisk tannbleking til hjemmebruk	NO	29
Kosmetisk tandblekning för hemmabruk	SV	32
Kosmetyczne wybielanie zębów w warunkach domowych	PL	35
Kosmetické bělení zubů pro domácí použití	CZ	38
Kozmetické bielenie zubov pre domáce použitie	SK	41
Albirea dinților prin tratament cosmetic la domiciliu	RO	44
Процедура косметического отбеливания зубов в домашних условиях	RU	47
Ev tipi Diş Beyazlatma	TR	50
تبييض أسنان تجميلي للاستخدام المنزلي	AR	53

Cosmetic tooth whitening for use at home

BlancOne® HOME+ synergistically combines the whitening effect of peroxides with an active, natural remineralization process using hydroxyapatite (HAP), which prevents the development of sensitivity.

The gel obtained by combining two ingredients, a whitening gel and an activator/remineralizing gel, using the self-mixing tip, is placed inside customized trays, which are then worn by the patient in the home treatment.

BlancOne® HOME+ can also be used for the internal whitening of non-vital teeth using the “walking bleach” technique.

BlancOne® HOME+ is available in 3 formulations:

- **BlancOne® HOME+ FAST**
6% HP Hydrogen peroxide + 0.5% HAP
Fast daytime whitening (45 mins.)
- **BlancOne® HOME+ NIGHT**
12% Carbamide peroxide (4.20% HP) + 0.5% HAP
Nighttime whitening
- **BlancOne® HOME+ EVER**
7% Carbamide peroxide (2.45% HP) + 1.0% HAP
Nighttime whitening recommended for sensitive teeth, maintenance, and orthodontic aligners

A. SESSION IN THE DENTAL PRACTICE

1. Before taking the whitening treatment it is essential to have a thorough examination to make sure that the soft tissues are healthy and identify any defective repair work or decay, which must be suitably treated.
2. If the soft tissue is traumatized, wait for it to heal before starting the whitening treatment.
3. Any tartar deposits or surface stains must be removed in an appropriate dental hygiene session.
4. Take an impression and create the individual trays. Trays with no reservoirs are recommended.
5. Observe the initial color and record it on the patient's record card or the digital file in the BlancOne® PRO app.
6. Show the patient the correct procedures for applying the whitening gel, positioning the filled trays in the mouth, the length of time spent on application, and the cleaning and storage of individual trays.
7. Give the patient the materials for home treatment and schedule a follow-up appointment.
8. During the follow-up appointment, check the condition of the oral tissues, observe the final color and record it on the patient record card or in the BlancOne® PRO App. The professional should discuss the results, and whether or not to continue with a further course of treatment, with the patient.

B. HOME WHITENING COURSE

BlancOne® HOME+ treatment requires daily application of the gel using customized trays.

The duration of treatment may vary according to clinical needs, treatment goals, and whether it is combined with treatment in the dental practice.

Each BlancOne® HOME+ syringe contains enough gel for 10-15 daily applications, depending on how much is used.

Each course of treatment requires at least 10 applications over a period of 2 weeks. Discontinuing the bleaching for a few days will not compromise the whitening effect when the treatment is resumed.

C. PREPARATION OF THE SYRINGE AND STORAGE OF THE GEL

1. Remove the cap from the syringe by turning it anti-clockwise.
2. Insert the automix tip and turn it clockwise to lock it.
3. The automix tip will remain attached to the syringe after the initial application.
4. NIGHT/EVER: Store the syringe at room temperature, away from light and heat sources, until the end of the course of treatment.
5. FAST: Keep the syringe refrigerated.

D. APPLICATION OF THE GEL

1. Apply a small amount of gel to the anterior (buccal) section of the trays, approximately mid-height, from premolar to premolar.
2. A small drop of gel - the size of a grain of rice - is enough for each tooth in the upper arch and every two teeth in the lower arch. Larger quantities will create wasted material that will leak out of the trays.
3. Dry the surface of the teeth with a cloth.
4. Insert the loaded trays, remove excess gel and leave in place for:
 - **45 minutes** - in the case of the BlancOne® HOME+ **FAST** rapid daytime treatment
 - **All night long** - if the BlancOne® HOME+ **NIGHT** or **EVER** nighttime treatment is carried out.
5. Remove the trays, then clean any remaining gel from the teeth and trays using a brush with water.
6. Leave the trays to dry and place them in the appropriate holder box.

WARNINGS FOR THE EUROPEAN UNION

- The mixed gel contains:
 - FAST - 6% Hydrogen Peroxide
 - NIGHT - 4.2% Hydrogen Peroxide in the form of 12% Carbamide Peroxide
 - EVER - 2.4% Hydrogen Peroxide in the form of 7% Carbamide Peroxide

- Not to be used in patients under 18 years.
- To be sold to dentists/orthodontists only.
- The first application of each course of treatment must be carried out by a dentist, or under direct supervision if this guarantees the same level of safety. Only then can it be made available to the patient, to complete the course of treatment.

CAUTION

- Keep out of the reach of children.
- Contains peroxide; ingesting large amounts may be hazardous. In the event of ingestion, seek medical advice.
- Avoid contact with the eyes. In the event of contact, rinse immediately with plenty of water.
- The mouth and gums must be in good condition before treatment begins. A dental examination is required before bleaching.
- The treatment must not be performed on patients with allergies or sensitization to one of the ingredients in the product.
- Not suitable for pregnant women. Seek medical advice when breastfeeding.
- Some patients may experience gum sensitivity or irritation. The effects are usually mild and do not last long. If the discomfort persists, discontinue the treatment and make an appointment to have the problem assessed.
- The treatment will not whiten crowns, inlays, fillings or other reconstructions.
- Internal whitening of non-vital teeth using the “walking bleach” technique must be carried out by a specialist following an endodontic procedure, an x-ray examination or restoration of the pulp chamber with careful preparation using glass ionomer cements, to avoid internal resorption and the leakage of oxidizing agents into the gutta-percha.

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