Geistlich Bio-Oss®

Geistlich Bio-Oss Pen®

Geistlich
Biomaterials

 **Manufacturer**
Geistlich Pharma AG
Bahnhofstrasse 40
6110 Wolhusen
Switzerland
Tel.: +41 41 492 55 55
info@geistlich.com
www.geistlich-pharma.com

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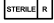
CE Mark
0123

Composition

Geistlich Bio-Oss® is a natural bone mineral of bovine origin. Geistlich Bio-Oss® is available as granules of spongy bone and as a spongy bone block. The granules are also available in an applicator as Geistlich Bio-Oss Pen®. Because of its natural origin, Geistlich Bio-Oss® is chemically as well as structurally comparable to the human bone mineral.

Sterilization of Geistlich Bio-Oss® is carried out by γ -irradiation. 

Properties / Effects

Geistlich Bio-Oss® has a macro- and microporous structure similar to human spongy bone. Geistlich Bio-Oss® supports the in-growth of new bone at the implantation site by serving as a stable scaffold for osteoblasts; over time it is subject to the physiological remodeling process. Geistlich Bio-Oss® is slowly resorbable, depending on the physiological environment of the individual patient, or integrated into the host bone. The highly purified osteoconductive mineral structure is produced from natural bone in a multistage purification process, adhering to the strictest safety regulations. Due to its properties, Geistlich Bio-Oss® is a proven alternative to autologous bone when used for bone regeneration in oral-maxillofacial areas. Geistlich Bio-Oss® is radiopaque like any phosphocalcic bone substitute.

Intended use

Geistlich Bio-Oss® / Geistlich Bio-Oss Pen® is intended to be used in the treatment of hard tissue defects in oral-maxillofacial surgery.

Geistlich

Biomaterials

 **Manufacturer**
Geistlich Pharma AG
Bahnhofstrasse 40
6110 Wolhusen
Switzerland
Tel.: +41 41 492 55 55
info@geistlich.com
www.geistlich-pharma.com

Indications

Geistlich Bio-Oss® / Geistlich Bio-Oss Pen® is indicated in the following clinical situations:

- > bone deficiencies in the alveolar ridge
- > dentofacial anomalies and malformations.

Geistlich Bio-Oss® block is especially recommended for filling of large bone deficiencies.

Instructions for use

Geistlich Bio-Oss® / Geistlich Bio-Oss Pen® should only be used by licensed dentists / physicians familiar with bone augmentation procedures. The general principles of sterile handling and patient medication must be followed when using Geistlich Bio-Oss® / Geistlich Bio-Oss Pen®.

Prior to the application of the device, ensure complete elimination of granulation tissue following exposure of the defect.

Application:

- Geistlich Bio-Oss Pen®: please see pictograms on the package label or on this sheet to follow the instruction below.
 - Hold the Geistlich Bio-Oss Pen® firmly: tear off the protective seal (A) and unscrew the protection cap (B). This exposes the applicator and filter cap.
 - Do not remove the red / green filter cap from the applicator at this time.
 - For complete wetting of Geistlich Bio-Oss®, hold the applicator with both hands and place the filter cap into the sterile and non-pyrogenic physiological saline solution or the non-coagulated patient blood. Completely retract and depress the plunger, using thumb pressure until Geistlich Bio-Oss® is completely moistened.
 - Once completely wetted, gently depress the plunger to eject any excess fluid.
 - Remove the filter cap from the applicator (A). Attach the curved applicator tip by screwing it on to the end of the applicator (B). The applicator is now ready for use. Depress the plunger and apply Geistlich Bio-Oss® directly to the surgical site.
- Geistlich Bio-Oss® granules in vials: mix Geistlich Bio-Oss® granules with the patient's blood or with sterile and non-pyrogenic physiological saline solution before the implantation. Granules are placed into the defect, using sterile instruments.
- Geistlich Bio-Oss® block: the block may be adapted to the desired size with a sterile scalpel and placed into the bone cavity. The Geistlich Bio-Oss® block can be applied dry or moistened with the patient's blood or with sterile non-pyrogenic physiological saline solution. In situ, the Geistlich Bio-Oss® block should be soaked with the patient's blood.
 - > In situ modeling may be performed with a sterile spatula or other suitable instrument.
 - > To minimize the risk of wound dehiscence or displacement of Geistlich Bio-Oss®, it is advisable to stabilize or to cover the treated defect (e.g., with Geistlich Bio-Gide® or Geistlich Mucograft® Seal).
 - > Local use of Geistlich Bio-Oss® / Geistlich Bio-Oss Pen® with medicinal products, alcohol, disinfectants or antibiotics has not been studied.

Special instructions for use in periodontology

A basic requirement for successful periodontal treatment includes eradicating the underlying bacterial infection as well as adequate oral hygiene. Therefore, prior to surgical intervention,

patients must receive a hygiene phase treatment, consisting of oral hygiene instructions, scaling and root planing, and occlusal adjustment when indicated.

Besides plaque control, the filling of periodontal defects with Geistlich Bio-Oss® requires successful local treatment of the periodontal lesion (root planing, debridement), prior to their-plantation.

Post-operative care

The usual post-operative care, medication and instructions should be given to patients.

Limitation of use / Precautions

Contraindications

Geistlich Bio-Oss® / Geistlich Bio-Oss Pen® must not be used in the presence of acutely infected wounds.

Precautions

Geistlich Bio-Oss® / Geistlich Bio-Oss Pen® should be used with special caution in patients with:

- > chronic infection (e.g. osteomyelitis) at the surgical site
- > uncontrolled metabolic disorders e.g. diabetes, osteomalacia, thyroid disorder, severe renal or hepatic disorder
- > prolonged corticosteroid therapy
- > immunological impairment diseases
- > radiotherapy
- > heavysmoking
- > or any medications that negatively influence bone healing.

To support sufficient regeneration and integration of bone, Geistlich Bio-Oss® should only be implanted in vital bony tissue and in direct contact with the host bone (if necessary with microfracture of the bone surface). For larger defects, the addition of autologous spongi-ous bone or bone marrow may improve the regeneration process.

Mechanical loading and insertion of implants (two-stage procedure) in the augmented area should be avoided until several weeks after the application of Geistlich Bio-Oss®. The appropriate timing of implant insertion depends on the residual local bone volume.

Intraoperatively, if there is need to remove the product, complete removal can be achieved. In the post-operative phase, complete removal may not be possible since the product is intended to resorb over time and will be integrated into the host bone.

Geistlich Bio-Oss® is a typical non-metallic diamagnetic material, in common with most human tissues. It cannot be heated or act as a magnet during the MRI examination. Geistlich Bio-Oss® has not been specifically studied in the MR environment.

The product is intended for use in a single patient in a single medical procedure. Any product not required in the single procedure should be discarded since sterility and stability cannot be assured once the original primary packaging is opened.

Do not use disinfectant or alcohol to clean the Geistlich Bio-Oss Pen® applicator prior to the use.

Side effects

Potential complications which may occur with any surgery include transient discomfort or pain, swelling at the surgical site, flap sloughing, bleeding, wound dehiscence, local inflammation, infection, bone loss, or dental implant loss.

Potential complications of an augmentation procedure may include material displacement, fast material resorption, insufficient material integration or bone formation.

Incompatibility reactions with Geistlich Bio-Oss® / Geistlich Bio-Oss Pen® cannot be totally excluded.

Pregnancy / Lactation

Data on the use of the product during pregnancy or lactation and in children before skeletal maturity are not available.

Other Information

Do not re-use

Do not resterilize

Use-by date

Temperature limit

Keep dry

Do not use if package is damaged

Keep away from sunlight

Caution

Consult electronic instructions for use

Sterilized using irradiation

Batch code

Catalogue number

Recycling symbol

Russian import symbol

Ukrainian registration symbol

 Authorized representative in the European Community / European Union

 Importer

 Sterile Barrier (Geistlich Bio-Oss®)

 Sterile Barrier (Geitlich Bio-Oss® Pen)

Prior to use, the integrity of the sterile barrier system must be visually inspected for breaches.
Do not use the product if packaging is damaged or opened unintentionally.

Presentation

Geistlich Bio-Oss Pen®:

Granule size: 1 – 2 mm

Content: 0.5 g (ca. 1.5 cm³)

Granule size: 0.25 – 1mm

Content: 0.25 g (ca. 0.5 cm³) and 0.5 g (ca. 1 cm³)

Geistlich Bio-Oss® granules:

Granule size: 1 – 2 mm

Vials of 0.5 g (ca. 1.5 cm³), 1.0 g (ca. 3 cm³) and 2.0 g (ca. 6 cm³)

Granule size: 0.25 – 1 mm

Vials of 0.25 g (ca. 0.5 cm³), 0.5 g (ca. 1 cm³), 1.0 g (ca. 2 cm³) and 2.0 g (ca. 4 cm³)

Geistlich Bio-Oss® block:

Block size: 1 cm x 1 cm x 2 cm

Patient Implant Card

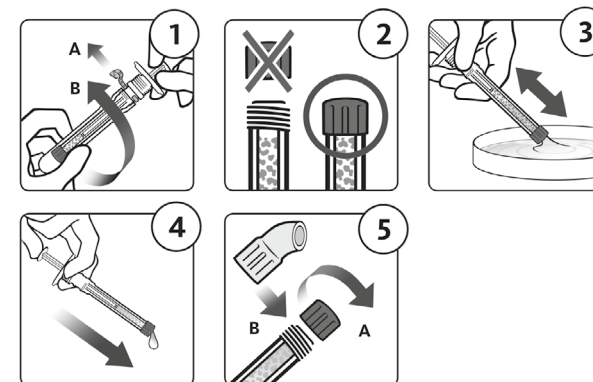
All information in regard of the provided PIC can be accessed under the following URL:
pic.geistlich-pharma.com

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